

Resusci Anne Simulator Info Sheet



Lubricate airway tools before insertion

Airway

- BVM with chest rise
- Oropharyngeal airway (OPA), nasopharyngeal airway (NPA), supraglottic insertion
- Suctioning
- Endotracheal intubation
- Lung sounds available (no chest rise)

Defibrillation

- Live defibrillation (**Shocklink only**)

ECG Connectors

- Can attach monitor to limb lead posts for 3-lead ECG

IM Injection Sites

- Air only

Eyes

- Gently open eyelids and manually swipe eyes up & down to select pupil size



No cutting or surgical procedures

Cardiac

- Instructor-view QCPR through SimPads/tablets/iPads
- ECG monitoring with a library of common ECGs (all ACLS rhythms)

Pulses

- Bilateral carotid pulses, right brachial, and right radial pulse (activate with palpation)
- **Do not press forcefully when palpating pulse points or no pulse will be felt**



Do not push or inject any fluids

IO Site Tibia

- Can perform drilling
- **No fluids**



Last updated: June 2023